
Alc Alert

Breath Alcohol Detector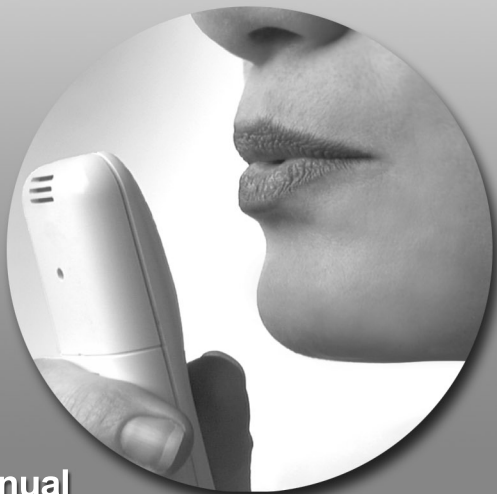

Operator's Manual
Bedienungsanleitung
Manuel de l'utilisateur
Manuale di istruzioni
Manual de instrucciones

BT5500

English.....	1
Deutsch	6
Français.....	11
Italiano	16
Español.....	21

Because of the variables involved in the timing of alcohol consumption, individual metabolism, and self-administration, the user of this product agrees (1) not to hold the manufacturer, dealer, wholesaler, or distributor responsible for the consequences of any decision, based on the use of this product, to operate a vehicle, boat, or aircraft or other equipment and (2) to hold them harmless from the claims of others arising out of any such decision. **IF YOU DO NOT AGREE, YOU SHOULD PROMPTLY RETURN THIS PRODUCT TO THE PLACE OF PURCHASE, UNUSED.**

Intended Use

The AlcAlert Breath Alcohol Detector is a screening test device for the rapid detection of alcohol in the breath. The test is used to obtain a visual quantitative result that approximates the level of alcohol concentration in the blood stream, and is intended for over the counter sale to the home user.

BEFORE USING THE UNIT

The AlcAlert™ Breath Alcohol Detector uses advanced semi-conductor gas sensor technology to approximate the percent of blood alcohol concentration (%BAC) from your breath. Please take a few moments to review the entire manual before using the unit.

Before using the AlcAlert Breath Alcohol Detector:

- When inserting batteries (not included with all models), please note the correct polarity on the inside of the battery case. Inserting the batteries with the incorrect polarity may damage the unit.


- Do not blow smoke or fluids into the mouth vent, as this may damage the sensor. Similarly, certain gases such as silicone compounds, sulfur and freon gases can also damage your sensor. Keep the unit away from all liquids and store in a safe environment.
- If the AlcAlert needs to be cleaned, wipe gently with a soft cloth. Do not use any cleaning agents.

AlcAlert is a trademark of PNI Corporation.

BEFORE USING THE UNIT

- Allow 30 minutes after consuming alcohol or smoking before using the unit. This helps to ensure the breath sample is taken from your lungs, and not from the alcohol within your mouth, which may result in a higher reading. In some cases, it may take up to one hour to reach the highest blood alcohol level after the last consumption of alcohol.
- Each individual's body responds to the consumption of alcohol differently. Consequently, a person's ability to drive safely may be impaired at a point well below the legal blood alcohol limit.

USING THE UNIT

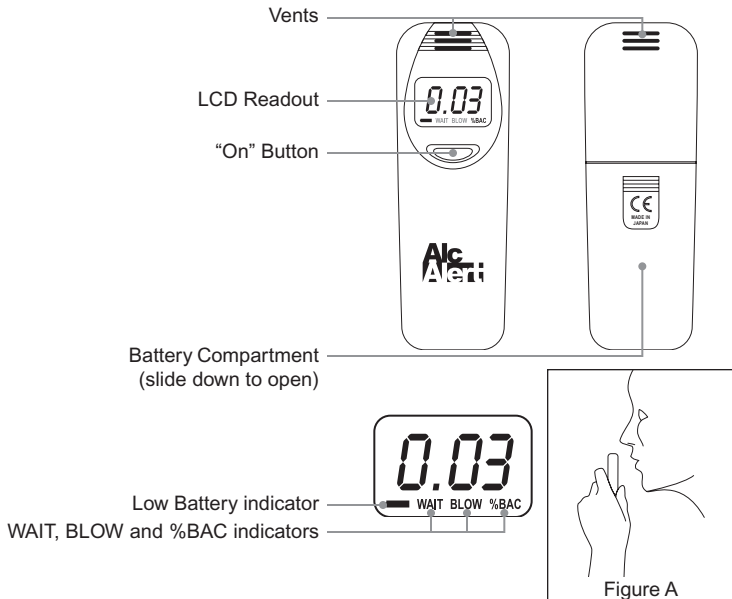1. Press the “On” button. The unit beeps once, the “WAIT” indicator flashes, and the unit begins a “warm-up” countdown from 14-to-0 or 6-to-0 (see Additional Information). You must wait until the countdown is complete. Proceed with step 2 only when the unit beeps twice and the “BLOW” indicator flashes.

Note: If “SAT” (saturation) flashes after the countdown is completed and the unit turns off, the sensor is saturated. Simply reactivate the unit to perform another warm-up countdown cycle. If “SAT” appears again, repeat this process as many times as necessary until “SAT” no longer appears. In most instances, 2 or 3 reactivation cycles should clear the “SAT” message. More reactivation cycles may be required if saturation is extensive (see Additional Information).

2. Blow into the vent. Blow continuously for about 5 seconds, until the unit beeps 3 times before displaying the results. Immediately as you start to blow, “Err” (Error) flashes until the unit beeps 3 times. Blow from between your lips, as if gently cooling a spoonful of soup. A distance of about half an inch to one inch between the vent and the mouth is suggested. (See figure A.)

Note: If you do not apply any breath at all, or if you stop blowing before you hear 3 beeps, the LCD flashes “Err” to indicate that you did not apply any or sufficient breath through the vent. If so, ignore the results and repeat the process.

3. Read the results. The %BAC results are displayed on the LCD display for about 5 seconds after the reading is taken. The results then flash for a few seconds until the unit turns off automatically.

The results are shown in increments of 0.01, ranging from 0.00 to 0.12. Results greater than 0.12 %BAC are displayed as “+.12”.

Understanding the results

The BAC (Blood Alcohol Concentration) represents the amount of ethyl alcohol in a given volume of blood. For instance, having a %BAC of 0.10 means that a person has 1 part alcohol per 1,000 parts blood in the body.

The most commonly used measurements are grams of ethanol per millilitre of blood (g/ml), used in the United States of America (also expressed as %BAC) and milligrams of ethanol per millilitre of blood (mg/ml), used in much of Europe (also expressed as promille).

For example, a reading of 0.05 %BAC on the AlcAlert Breath Alcohol Detector is the same as 0.05 g/ml or 50 mg/ml or 0.5 g/l or 0.5 promille.

ADDITIONAL INFORMATION

- The normal warm-up countdown is from 6-to-0. However, if it has been more than 1 hour since the last operation, the unit begins a 14-to-0 warm-up countdown.
- The sensor must be warmed up properly to report accurate readings. Therefore, always disregard the results following the initial 14-to-0 warm-up countdown.
- The “SAT” (saturation) message feature prevents the unit from displaying inaccurate %BAC readings in the case of sensor saturation by contaminating agents. For example, saturation often occurs when excessive alcohol moisture is deposited on the sensor from multiple consecutive uses, or by the accumulation of air contaminants on the sensor after a long period of inoperability, such as upon first use. In such instances, the unit requires additional warm-up cycles to properly recondition the sensor.
- If the low battery indicator flashes then remains on during use, replace the batteries.
- The AlcAlert Breath Alcohol Detector Model BT5500 includes a backlight. The backlight turns on and off automatically.
- Always blow into the unit in a consistent manner. Different blowing conditions may produce different results.
- Your unit is more accurate when your true blood alcohol concentration is between 0.02 %BAC and 0.08 %BAC and less accurate as your blood alcohol concentration is over 0.08 %BAC.
- The “SAT” message feature prevents the unit from displaying inaccurate %BAC readings in the case of sensor saturation by contaminating agents. For example, saturation often occurs when excessive alcohol moisture is deposited on the sensor from multiple consecutive uses, or by the accumulation of air contaminants on the

sensor after a long period of inoperability, such as upon first use. In such instances, the unit requires additional warm-up cycles to properly recondition the sensor.

Each individual's body responds to the consumption of alcohol differently. Consequently, a person's ability to drive safely may be impaired at a point well below the legal blood alcohol limit.

The AlcAlert Breath Alcohol Detector should be used only to give an indication of the possible presence of alcohol in the blood. You should not rely upon it as the sole basis to determine intoxication or whether it is safe to drive a vehicle, operate equipment, or engage in dangerous activities.

PNI Corporation does not recommend drinking and driving. If you are going to drink alcohol, always have a non-drinking designated driver. Do not drink and drive!

- **Dimensions:**
90 mm x 31.5 mm x 16.5 mm; 50 grams
 - **Sensor:**
Semiconductor gas sensor
 - **Accuracy*:**
±20% at 0.08 %BAC for two years, under normal operating conditions and factors**
 - **Display:**
%BAC in 0.01 increments ranging from 0.00 to 0.12
 - **Battery life:**
2 AAA alkaline batteries for over 300 uses
 - **Product lifetime:**
Two years, under normal operating conditions and factors
 - **Temperature range:**
-20°C to +70°C (storage); 0°C to +40°C (operating)
- * *Accuracy varies depending on individual metabolism, blowing conditions, humidity, and the condition of the sensor.*
- ** *Normal operating conditions and factors means an ambient temperature range of 10° to 35°C; a relative humidity range of 40% to 60%; a clean air environment (no contamination or poisoning agents), and normal usage (fewer than 20 operations per day).*

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PNI Corporation • 5464 Skylane Blvd. • Santa Rosa, CA 95403 • www.pnicorp.com